



Product Guide for Veterinary Clinics

Designed for oesophagostomy tube comfort and recovery

FeliFeed® Support Collars offer a washable, eco-conscious alternative to bulky bandages — helping protect oesophagostomy feeding tubes while reducing daily vet visits and complex dressing changes. Designed for cats (or small dogs) recovering at home, each collar supports short-term healing with comfort and dignity, easing stress for both pets and carers.

Made in the UK by Mabel Green Studio using sustainable materials, each pack includes:

- One collar (individually packaged in a recyclable, plastic-free bag)
- A step-by-step illustrated guide for carers

Safe to leave in place during radiographs, chemotherapy, and other vet procedures. We recommend placing a collar post-surgery and offer clients a second collar for home use — one to wear, one to wash. **For additional carer's guidance, including a visual fitting overview, please see page 2.**

 Design Features	 Clinical Benefits
<ul style="list-style-type: none">Made with locally sourced, sustainable fabricsMachine washable up to 30 °C (gentle cycle)Adjustable Velcro® closure ensures a secure, comfortable fitReinforced buttonhole allows easy tube placement and accommodates most types, including Y-portsDiscreet loop at the base for attaching a cat's ID tag or small bellIndividually wrapped in recyclable, plastic-free packaging 	<ul style="list-style-type: none">Supports short-term recovery at homeMinimises stress for carers and cats by simplifying daily care routinesRemoves the need for complex dressings and bulky bandagesReduces skin irritation and tube movementDesigned for comfort during supervised wearSafe to leave in place during radiographs, chemotherapy, and other vet procedures

Collar Sizes	Weight	Neck Size	Collar Length + Tab*	Collar Depth
Extra-Petite: Kittens	600g – 2kg	12 – 18 cm	20 + 5 cm	3 cm
Petite: Petite breeds such as Cornish Rex or Devon Rex	2.5 – 3.5 kg	16 – 22 cm	22 + 5 cm	4 – 4.5 cm
Small: Slim adult cats with short to medium fur (typical Domestic Shorthair body type)	3.6 – 5.5 kg	23 – 25 cm	28 + 5 cm	4.5 cm
Medium: Larger domestic shorthair or smaller fluffy breeds	5 – 7.5 kg	25 – 33 cm	34 + 5 cm	5 cm
Large: Large or fluffy breeds such as Maine Coon or Norwegian Forest Cat	7.1 – 9.5+ kg	28 – 40 cm	38 + 8 cm	5.5 – 6 cm

* The Velcro tab fastener allows the collar length to be adjusted/shortened, to fit the cat's neck.


 Sizing & Custom Orders	 Price & Ordering
Each FeliFeed® Support Collar is hand-cut and crafted at Mabel Green Studio. As a result, there may be slight variations in the finished size. If you require a custom size, please don't hesitate to get in touch to discuss your patient's needs. Email: hello@mabelgreenstudio.co.uk	<ul style="list-style-type: none">Prices: X-Petite £20, Petite £24, Small £26, Medium £28, Large £30Dispatch: Typically, within 1–2 working days if in stock; allow up to 5 days for larger or custom ordersPostage: £3.95 Royal Mail Tracked 48, or £4.95 Tracked 24Free standard shipping on orders of £100 or more

Availability & Official Channels: FeliFeed® Support Collars will be available to buy directly from **Mabel Green Studio** — website going live soon! In the meantime, orders can be made, or quotations provided, by emailing Louise at hello@mabelgreenstudio.co.uk **Please note:** We do not work with unsolicited resellers. **Prices are subject to change.**

 **Clinician endorsed by Manchester Cat Clinic**

"Purr-fect for our patients ... soft, well-tolerated, and exactly the right design."

Ellie Lee BVSc CVLM MRCVS, Director

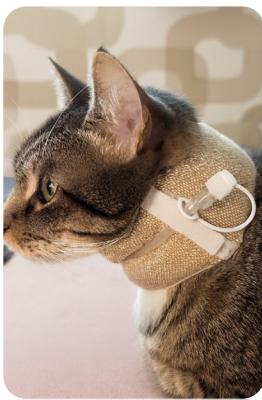

Additional Support for Carers

For a helpful visual overview, we recommend:

"A Guide to Caring for a Cat with an Oesophagostomy Tube"

Produced by iCatCare, an international charity dedicated to feline health and welfare.

Especially helpful for carers who are new to stoma care, this video provides practical guidance on feeding, hygiene, and comfort — including how to fit the collar correctly.

Watch on YouTube: bit.ly/4769FhI or scan the QR code.



Return Policy

We want you to feel confident and supported when ordering FeliFeed® Support Collars.

- If you mistakenly order the wrong size, we're happy to offer an exchange — provided the packaging remains sealed and the collar is unused.
- If a product arrives damaged, faulty, or not as described, you are entitled to a full refund or replacement. Please contact us within 14 days of receipt to arrange this.
- As a clinical-use item, we're unable to accept returns once the packaging has been opened, unless the item is defective.

This policy complies with the Consumer Rights Act 2015 and UK distance selling regulations.

For all enquiries, please email hello@mabelgreenstudio.co.uk and we'll be happy to help.

FeliFeed Support Collar® is a registered trademark.

All designs, text, and packaging are copyright protected.

© Mabel Green Studio 2025

Unauthorised reproduction, resale, or distribution is strictly prohibited.



Clinician endorsed by Manchester Cat Clinic

"Purr-fect for our patients ... soft, well-tolerated, and exactly the right design."

Ellie Lee BVSc CVLM MRCVS, Director